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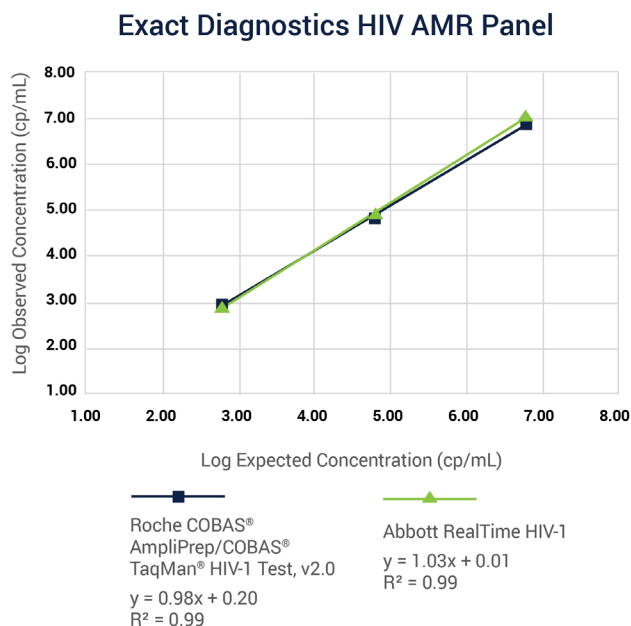
Tri jedinstvene serije proizvoda EDX HIV-1 AMR Panel testirane su u triplikatu u testovima navedenim na grafičkom prikazu u nastavku.

The data provided is for informational use only. Expected results must be established by the end user for the particular assay system used.

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-  Kataloški broj
-  Broj serije
-  Za in vitro dijagnostičku primjenu
-  Biološki rizici
-  Datum isteka valjanosti
-  Nemojte ponovno upotrijebiti
-  Gornja temperaturna granica
-  Proizvođač
-  Oprez
-  Pogledajte upute za uporabu

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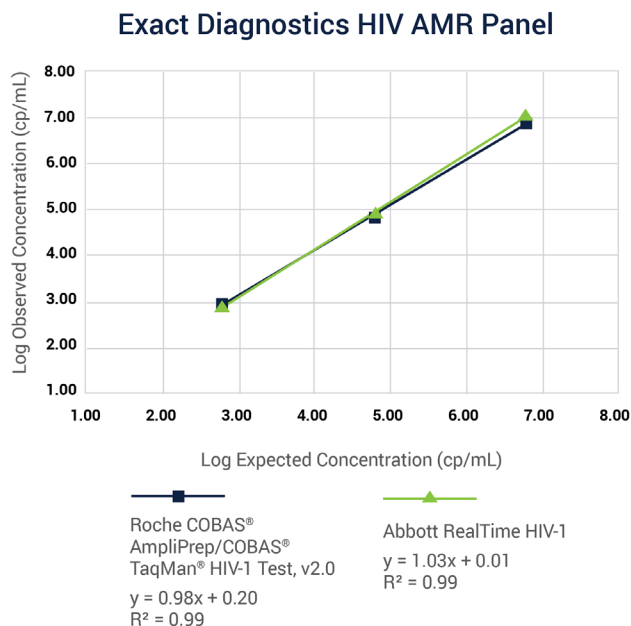
²Siegel JD, Rhinehart E, Jackson M, Chiarello L, and the Healthcare Infection Control Practices Advisory Committee, 2007 Guideline for Isolation Precautions: Preventing Transmission

Exact Diagnostics LLC
3400 Camp Bowie Blvd CBH-214
Fort Worth TX 76107
SAD

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ODKAZY:

¹The 3rd WHO International Standard for HIV-1 (10/152) IFU.
<http://www.nibsc.org/documents/ifu/10-152.pdf>

² Siegel JD, Rhinehart E, Jackson M, Chiarello L, and the Healthcare Infection Control Practices Advisory Committee, 2007
Guideline for Isolation Precautions: Preventing Transmission

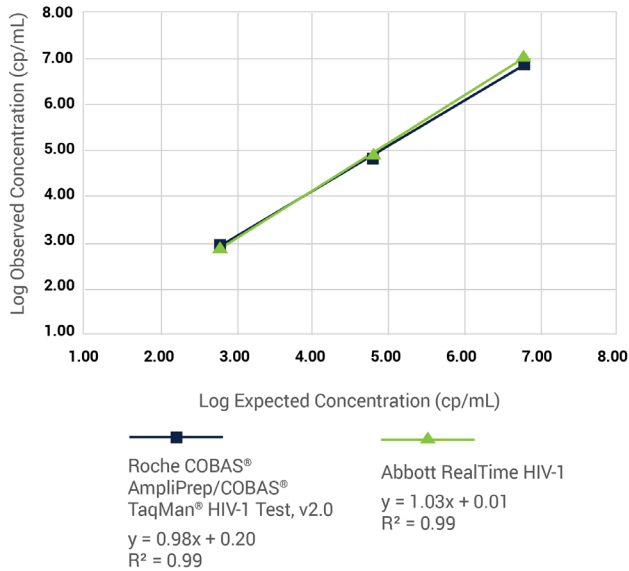
Exact Diagnostics LLC
3400 Camp Bowie Blvd CBH-214
Fort Worth, Texas 76107
Spojené státy

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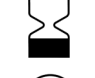
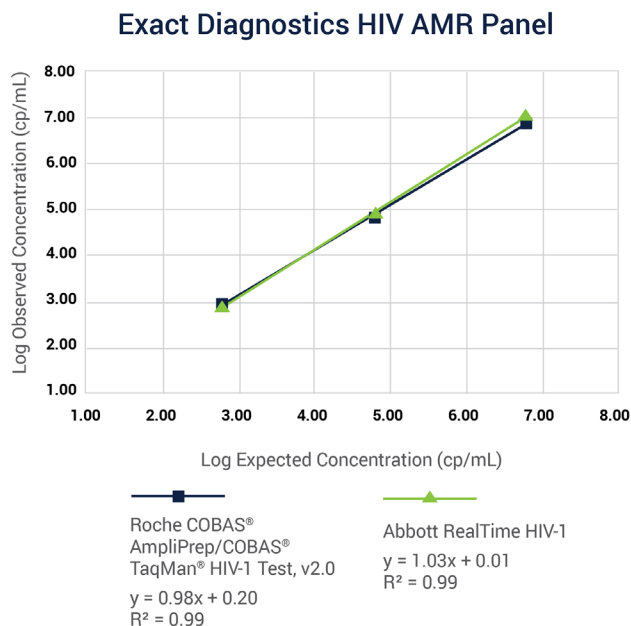
Tutvuge kasutusjuhistega

VIITED

¹ The 3rd WHO International Standard for HIV-1 (10/152) IFU.
<http://www.nibsc.org/documents/ifu/10-152.pdf>

² Siegel JD, Rhinehart E, Jackson M, Chiarello L, and the Healthcare Infection Control Practices Advisory Committee, 2007 Guideline for Isolation Precautions: Preventing Transmission

Exact Diagnostics LLC
3400 Camp Bowie Blvd CBH-214
Fort Worth TX 76107
Ameerika Ühendriigid

Tootja: Exact Diagnostics LLC, 3400 Camp Bowie Blvd. CBH-214, Fort Worth, Texas 76107

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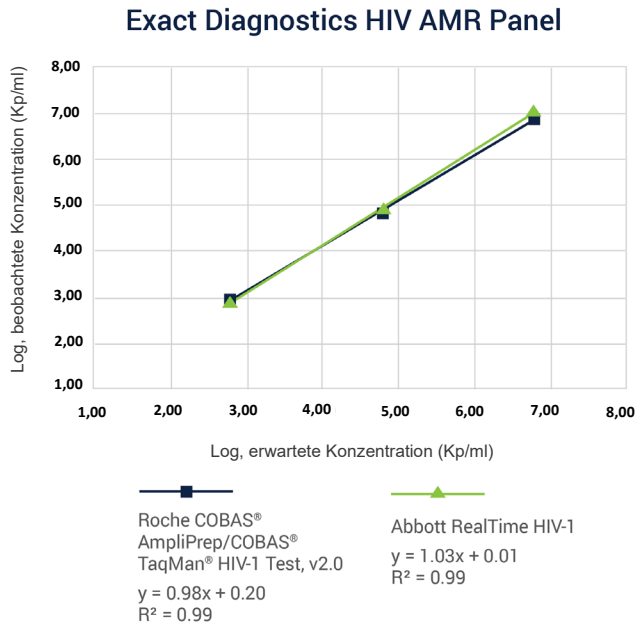
LITERATUR:

¹The 3rd WHO International Standard for HIV-1 (10/152) IFU.
<http://www.nibsc.org/documents/ifu/10-152.pdf>

²Siegel JD, Rhinehart E, Jackson M, Chiarello L, and the Healthcare Infection Control Practices Advisory Committee, 2007
 Guideline for Isolation Precautions: Preventing Transmission

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