

EDX RP Positive Run Control Summary	
Catalog Number	RPPOS
Analyte	Adenovirus <i>Bordetella parapertussis</i> <i>Bordetella pertussis</i> <i>Chlamydia pneumoniae</i> Coronavirus 229E Coronavirus HKU1 (RNA IVT) Coronavirus NL63 (RNA IVT) Coronavirus OC43 Human Metapneumovirus (RNA IVT) Influenza A – H1N1 Influenza A – H1N1 2009 (RNA IVT) Influenza A – H3N2 (RNA IVT) Influenza B <i>Mycoplasma pneumoniae</i> Middle Eastern Respiratory Syndrome (RNA IVT) Parainfluenza 1 Parainfluenza 2 Parainfluenza 3 Parainfluenza 4 Rhinovirus Respiratory Syncytial Virus A Respiratory Syncytial Virus B Severe Acute Respiratory Syndrome Coronavirus 2
Matrix	Synthetic Matrix
Preservative	ProClin® 300
Storage	-20°C or Below
Fill Volume	0.3mL
Number of Vials Per Kit	6 Vials
Number of Uses	Single Use
Shelf Life	18 Months from Date of Manufacture
Precautions	Biological Source Material. Treat as Potentially Infectious
Regulatory Status	For In Vitro Diagnostic Use

INTENDED USE:

The EDX RP Positive Run Control is an external quality control, intended to be used with respiratory assays to monitor the performance of that given assay. Routine use of EDX RP Positive Run Control allows laboratories to meet their individualized quality control plans including evaluation of reagent lot changes and new shipments, monitoring of multiple identical devices, and assessment of different personnel and locations.

PRODUCT DESCRIPTION:

The EDX RP Positive Run Control contains 23 respiratory analytes. This multiplex control is a combination of whole, intact virus and bacteria, that have been heat or chemically inactivated, and synthetic RNA transcripts. The multiplex control is intended to test the entire process of a molecular assay including extraction, detection, and amplification. EDX RP Positive Run Control is internally value assigned using Bio-Rad Digital Droplet PCR (ddPCR). EDX RP Positive Run Control is formulated in synthetic matrix which contains preservatives.

PROCEDURE:

The EDX RP Positive Run Control requires an extraction step. The control must be treated in a similar manner to other tested samples. This allows the operator to assess their extraction technique.

The EDX RP Positive Run Control is an unassayed control and does not have assigned values. The control is not a substitute for the manufacturer’s kit controls provided with the assay. EDX RP Positive Run Control is to be tested according to the assay manufacturer’s or testing laboratory’s instructions and recommendations. Expected results utilizing EDX RP Positive Run Control must be established by the end user for their specific assay.

STORAGE AND HANDLING:

The EDX RP Positive Run Control should be stored at -20°C or below.

Thaw EDX RP Positive Run Control at room temperature and vortex briefly prior to use.

Once thawed, EDX RP Positive Run Control must be used immediately. Any remaining materials must be disposed of after one use. Do not reuse. Do not dilute.

Do not use EDX RP Positive Run Control beyond the expiration date.

LIMITATIONS:

For In Vitro Diagnostic Use.

WARNINGS AND PRECAUTIONS:

The EDX RP Positive Run Control contains heat inactivated virus and bacteria but should be treated as potentially infectious. Universal precautions and proper disposal should be practiced¹. Do not pipette by mouth. Do not smoke, eat, or drink in areas where specimens are handled.

Discard product if packaging is damaged or leaking. Disinfect liquids, materials or spills with a 0.5% sodium hypochlorite solution. Dispose of all materials and liquids used in the procedure as if they contained pathogenic agents.

PERFORMANCE CHARACTERISTICS:

The data provided for EDX Positive Run Control when tested on the BioFire® FilmArray® Respiratory Panel (RP), BioFire® FilmArray® Respiratory Panel 2 (RP2), BioFire® FilmArray® Respiratory Panel 2 *plus* (RP2*plus*), BioFire® Respiratory 2.1 Panel (RP2.1), and the BioFire® Respiratory 2.1 *plus* Panel (RP2.1*plus*) is for informational use only. Expected results must be established by the end user for the assay system used. Results may vary depending on instrumentation, reagents, operators, and systematic or random errors.

BioFire® FilmArray® Respiratory Panel Results

	RP	RP2	RP2 <i>plus</i>	RP2.1	RP2.1 <i>plus</i>
Viruses					
Adenovirus	Detected	Detected	Detected	Detected	Detected
Coronavirus 229E	Detected	Detected	Detected	Detected	Detected
Coronavirus HKU1	Detected	Detected	Detected	Detected	Detected
Coronavirus NL63	Detected	Detected	Detected	Detected	Detected
Coronavirus OC43	Detected	Detected	Detected	Detected	Detected
Severe Acute Respiratory Syndrome	N/A	N/A	N/A	Detected	Detected
Coronavirus 2 (SARS-CoV-2)					
Human Metapneumovirus	Detected	Detected	Detected	Detected	Detected
Human Rhinovirus/Enterovirus	Detected	Detected	Detected	Detected	Detected
Influenza A – H1-2009	Detected	Detected	Detected	Detected	Detected
Influenza A – H3	Detected	Detected	Detected	Detected	Detected
Influenza B	Detected	Detected	Detected	Detected	Detected
Middle East Respiratory Syndrome	N/A	N/A	Detected	N/A	Detected
Coronavirus					
Parainfluenza 1	Detected	Detected	Detected	Detected	Detected
Parainfluenza 2	Detected	Detected	Detected	Detected	Detected
Parainfluenza 3	Detected	Detected	Detected	Detected	Detected
Parainfluenza 4	Detected	Detected	Detected	Detected	Detected
Respiratory Syncytial Virus	Detected	Detected	Detected	Detected	Detected
Bacteria					
<i>Bordetella parapertussis</i>	N/A	Detected	Detected	Detected	Detected
<i>Bordetella pertussis</i>	Detected	Detected	Detected	Detected	Detected
<i>Chlamydia pneumoniae</i>	Detected	Detected	Detected	Detected	Detected
<i>Mycoplasma pneumoniae</i>	Detected	Detected	Detected	Detected	Detected

REFERENCES:

¹ Siegel JD, Rhinehart E, Jackson M, Chiarello L, and the Healthcare Infection Control Practices Advisory Committee, 2007 Guideline for Isolation Precautions: Preventing Transmission

BIO-RAD and DROPLET DIGITAL PCR are trademarks of Bio-Rad Laboratories, Inc. in certain jurisdictions.



**Bio-Rad
Laboratories**

**Clinical
Diagnostics Group**

9500 Jeronimo Road
Irvine, California 92618
(800) 854-6737
FAX (949) 598-1550
bio-rad.com/qualitycontrol

Technical Service:
(800) 854-6737

United States Customer Service: 4000 Alfred Nobel Drive • Hercules, California 94547 • Phone (510) 724-7000 • FAX (510) 741-6373
Also in: **Gladesville, Australia,** Phone +61 (2) 9914 2800 • Fax +61 (2) 9914 2888 **Vienna, Austria,** Phone +43 (0) 1 877 89 01 9 • Fax +43 (0) 1 876 56 29 **Temse, Belgium,** Phone +32 (0) 3 710 53 00 • Fax +32 (0) 3 710 53 01 **São Paulo, Brazil,** Phone +55 11 3065-7550 **Montréal, Canada,** Phone +1 514 334 4372 • Fax +1 514 334 0872 **Shanghai, China,** Phone +86 21 6169 8500 • Fax +86 21 6169 8599 **Prague, Czech Republic,** Phone +420 241 431 660 **Symbion Science Park, Denmark,** Phone +45 44 52 10 00 • Fax +45 44 52 10 01 **Helsinki, Finland,** Phone +358 9 804 22 00 **Marnes-la-Coquette, France,** Phone +33 (0)1 47 95 60 00 • Fax +33 (0)1 47 41 91 33 **Munich, Germany,** Phone +49 (0) 89 31884 393 • Fax +49 (0) 89 31884 136 **Athens, Greece,** Phone +30 210 7774396 • Fax +30 210 7774376 **Quarry Bay, Hong Kong,** Phone +85 2 2789 3300 • Fax +85 2 2789 1257 **Budapest, Hungary,** Phone +36 1 459 6190 • Fax +36 1 459 6101 **Haryana, India,** Phone +91 124 4029300 • Fax +91 124 2398115 **Rishon Le Zion, Israel,** Phone +972 03 963 6025 • Fax +972 03 951 4129 **Milan, Italy,** Phone +39 02 94 86 600 • Fax +39 02 21609399 **Tokyo, Japan,** Phone +81 3 6361 7070 • Fax +81 3 5463 8481 **Seoul, Korea,** Phone +82 080 007 7373 • Fax +82 (2) 3472 7003 **Mexico D.F., Mexico,** Phone +52 (55) 5488 7670 • Fax +52 (55) 1107 7246 **Veenendaal, The Netherlands,** Phone +31 (0)318 540 666 • Fax +31 (0)318 542 216 **Auckland, New Zealand,** Phone +64 (9)415 2280 • Fax +64 (9)415 2284 **Oslo, Norway,** Phone +47 23 38 41 30 • Fax +47 23 38 41 39 **Warsaw, Poland,** Phone +48 22 331 99 99 • Fax +48 22 331 99 88 **Amadora, Portugal,** Phone +351 21 47 27 700 • Fax +351 21 472 7777 **Moscow, Russia,** Phone +7 495 721 1404 • Fax +7 495 721 1412 **Singapore,** Phone +65 6415 3170 • Fax +65 6415 3189 **Johannesburg, South Africa,** Phone +27 11 442 8508 • Fax +27 11 442 8525 **Madrid, Spain,** Phone +34 91 490 6580 • Fax +34 91 590 5211 **Sundbyberg, Sweden,** Phone +46 844 98053 • Fax +46 8 55 51 27 80 **Cressier, Switzerland,** Phone +41 (0) 61 717 9555 • Fax +41 (0) 61 717 9550 **Taipei, Taiwan,** Phone +886 (2) 2578-7189 • Fax +886 (2) 2578-6890 **Bangkok, Thailand,** Phone (662) 651 8311 • Fax (662) 651 8312 **Hertfordshire, United Kingdom,** Phone +44 (0)1923 471301 • Fax +44 (0)1923 471340

SYMBOL REFERENCES:



Catalog Number



Lot Number



For In Vitro Diagnostic Use



Expiration Date



Upper Limit of Temperature



Manufacturer



Caution




Consult Instructions for Use



Positive Control



European Mark of Conformity

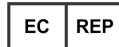

Authorized Representative



Emergo Europe
Prinsessegracht 20
2514 AP The Hague
The Netherlands



Exact Diagnostics LLC
100 South Jones Street, Suite 100,
Fort Worth, Texas 76104
United States